**seca 231/232**

<b>Deutsch</b>	<b>Seite 4</b>
<b>Englisch</b>	<b>Page 6</b>
<b>Français</b>	<b>Page 8</b>
<b>Español</b>	<b>Página 10</b>
<b>Italiano</b>	<b>Pagina 12</b>
<b>Dansk</b>	<b>Side 14</b>
<b>Svenska</b>	<b>Sida 16</b>
<b>Norsk</b>	<b>Side 18</b>
<b>Suomi</b>	<b>Sivu 20</b>
<b>Nederlands</b>	<b>Pagina 22</b>
<b>Português</b>	<b>Página 24</b>
<b>Ελληνικά</b>	<b>Σελίδα 26</b>
<b>日本語</b>	<b>28</b>
<b>Polski</b>	<b>Strona 30</b>

Montageanweisung

Assembly instructions

Instructions de montage

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Monteringshandledning

Montasjeveiledning

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Montagehandleiding

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## Congratulations!

In the seca 231/232 measuring rod, you have acquired an accurate and at the same time sturdy piece of equipment for measuring the length of babies.

The measuring rod serves both to determine the length at birth of a newborn and to check body length during the growth process.

It is attached to the following seca baby scales using the assembly material supplied:

Measuring rod 231:

model 717, 728, 736, 738 and 739.

Measuring rod 232:

model 334, 335 and 336

In combination with a scale, therefore, the measuring rod serves as a complete measuring and weighing station.

The measuring rod is manufactured from stable, easy-care aluminium profile. The head and foot supports supplied make measuring convenient and safe.

seca 231 and seca 232 measuring rods are for use with seca baby scales in hospitals and doctors' surgeries.

## Safety

Before using the new measuring rod, please take a little time to read the following safety instructions:

- Follow the operating instructions.
- Attach the measuring rod only to the seca baby scales listed above
- Use the assembly material supplied
- When assembling, ensure that screws

are tight and that the rod has been assembled correctly (see chapter entitled "Assembly instructions")

- Never leave a baby on the scale unsupervised.
- When sliding the measuring rod, ensure that neither the baby's hands or feet are on the measuring rod.
- Ensure that the measuring rod and scale are cleaned and disinfected regularly to prevent infectious diseases being transmitted.
- A second person is ideally required to enable length to be measured exactly.

## Assembly instructions


Page 3.


**Note:** the measuring rod is fitted to the tray in elongated holes to allow the measuring rod to be adjusted exactly and unavoidable production tolerances to be compensated. Place a known reference measure between the head and foot stops and move the bracket on the tray so that the correct length is displayed. In general, however, it is sufficient to fit the rod centrally in the elongated holes.

## Operating instructions

### How to measure properly...

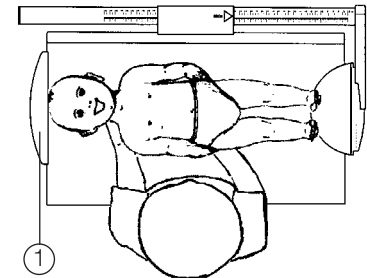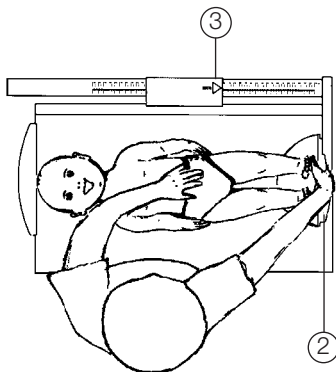1. Lay the baby on the weighing tray. The baby's head must be at the head stop (1) of the tray.
2. Using your right hand, push the foot support (2) in the direction of the baby's feet, placing the baby's feet on the bearing surface of the foot support. Keep the baby still on the weighing tray with the left hand.
3. With the left hand, gently press the baby's knees down onto the weighing tray and with the right, push the foot support (2) to the soles of the baby's feet. Read off the length of the baby from the marking on the measuring rule guide (3).

## Cleaning

Clean the measuring rod as required using a domestic cleaning agent or commercially available disinfectant. Follow the manufacturer's instructions.

## Technical data

Measurement range: 350-800 mm /  
14 - 32 inches  
Graduations: 1 mm / 1/8 inch  
Accuracy: greater than  $\pm 5$  mm  
Dimensions  
(W x H x D): 640 mm x 120 mm x  
276 mm  
Weight: approx. 2000 g  
Temperature range: +10 °C to +40 °C  
Medical product in  
accordance with  
Directive 93/42/EEC: Class I with measuring  
function

## Disposal

Should the measuring rod no longer be able to be used, the responsible waste disposal association will be glad to give you information about the measures necessary to dispose of it properly.

## Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the merchandise secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

 Declaration of conformity

Mechanical equipment for measuring  
length

seca model 231  
seca model 232

Batch no.: see back of measuring rod

The measuring rod meets the applicable requirements of Directive 93/42/EEC on medical products.

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